

Connecting ECG machine to ECG1000


I. Operation on the computer side:

Lookup of the IP address of a computer:

1. Open the Command Prompt: Press the Windows key (Windows logo key) + R, then type "cmd" and press Enter. This will open a Command Prompt window.
2. Enter the command: In the Command Prompt window, type "ipconfig" and press Enter. Please record the IP address (IPv4) of the computer displayed here.



II. Operation on the ECG machine side:

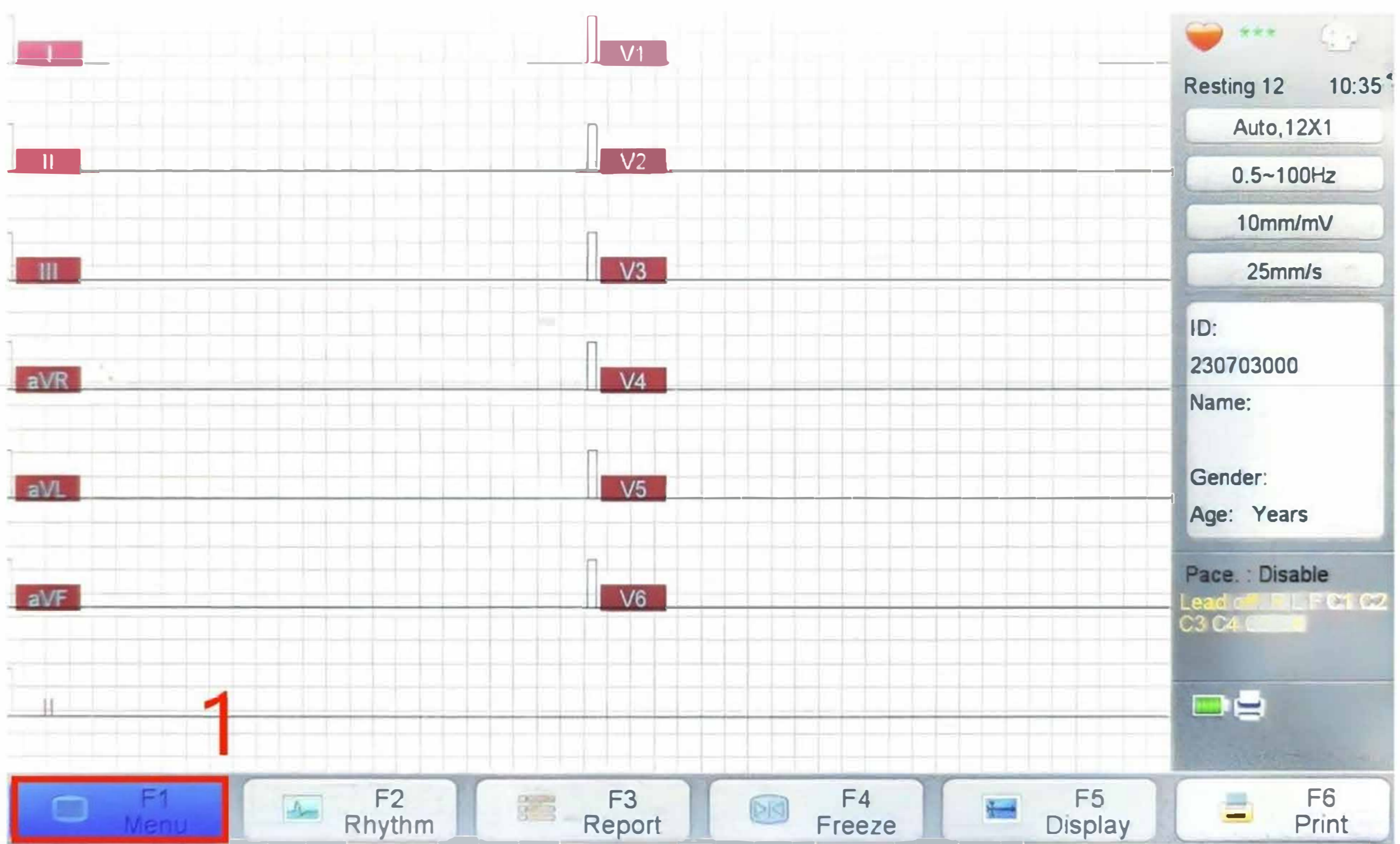

3 System Setting

Network Setting	Cable Network / [WiFi Network]	X
Transfer Protocol	IP Address	192.168.3.217
Cable Network	Subnet Mask	255.255.255.0
WiFi Network	Default Gateway	192.168.3.254
3G Network	OK	Cancel

2 System Setting

- ECG Setting
- Print Setting
- Display Setting
- Patient Information
- System Setting**
- Data Management
- APPT List
- Remote Report
- Factory Maintain

IP Address: Set an IP address for the ECG machine that is not occupied by a computer or other device.
Subnet Mask: Same value as computer setting.
Default Gateway: Same value as computer setting.

Or connect the ECG machine to a Wi-Fi network (2.4GHz only).

Press [Esc] to return previous menu.

Save Exit

4 System Setting

WiFi Network	Server Setting	X
3G Network	IP Address	IP address of the computer
Server Setting	Port	8000
FTP Server	OK	Cancel
Silent Mode		
QRS Tone	6	>
Alarm Tone	6	>
Key Tone	6	>
Default Memory	Internal Memory	>
Memory Format	Click to Enter	>

Press [Esc] to return previous menu.

Save Exit