

## Oxivir Plus Spray

### Oxivir Plus Spray: Broad Spectrum Cleaner & Disinfectant

#### Description

Diversey **Oxivir Plus Spray** is a ready-to-use liquid cleaner disinfectant for the cleaning and disinfection of all water-resistant hard surfaces and some soft furnishings.

#### Key properties

Hydrogen Peroxide based acidic formulation  
Effective against a wide spectrum of micro-organisms including bacteria, viruses, fungi and yeasts  
Effective cleaner  
Fragrance-free  
Based on patented technology; accelerated hydrogen peroxide

#### Benefits

Cleans and disinfects in one step, reduction of working time  
Extensively tested against European norms and proven broad spectrum disinfection:  
- Bactericidal, Virucidal, Fungicidal and Yeasticidal  
Excellent cleaning performance offering a streak free finish, descaling and lime limitation  
Suitable for use on water-resistant hard surfaces and some soft furnishings  
Good user safety profile  
Good environmentally profile. Active breaks down into oxygen and water  
Suitable for use in all kind of application areas, eg in high risk areas in Healthcare

#### Use instructions

Dosing:  
Use undiluted

Application:

Spray cleaning and disinfection:

1. Remove loose dirt
2. Spray onto surfaces
3. Clean with clean cloth, pre-wetted with product
4. Spray again for at least 5 minutes and wipe to dry

Important:

Do not mix with other products. Do not use on water- or acid sensitive surfaces.

#### Technical data

Appearance:	Clear colourless liquid
Relative density [20°C]:	1.000
pH-value in use:	1.8

The above data is typical of normal production and should not be taken as a specification

 **PRAXISDIENST**  
Medical Supplies since 1953

 **Order here!**



## Oxivir Plus Spray

### Safe handling and storage information

Full guidance on the handling and disposal of this product is provided in a separate Material Safety Data Sheet. Store in original container. Avoid extremes of temperature. Reacts with oxydisable substances.

### Product compatibility

Under recommended conditions of use Diversey **Oxivir Plus Spray** is suitable for use on most materials commonly encountered.

### Environmental information

The surfactants used in this product are ultimately biodegradable in compliance with the requirement of the EU Detergent Regulation, EC 648/2004.

### Microbiological data

Efficacy proven by compliance with: EN1276, EN13697, EN1650, EN14476, EN13624 and EN13727.

### Available pack sizes

Article code	Pack size	Pack type
7519553	<b>Oxivir Plus Spray</b> 6x0.75L	Spray bottle with foam trigger
7519554	<b>Oxivir Plus Spray</b> 6x0.75L	Spray bottle with foam trigger
7519559	<b>Oxivir Plus Spray</b> 6x0.75L	Spray bottle with foam trigger

### Diversey Europe Operations B.V.

De Boelelaan 32  
1083 HJ Amsterdam  
The Netherlands  
Tel +31 (0)20 258 9700  
[www.diversev.com](http://www.diversev.com)