



URINOCOL®

PEADIATRIC URINE BAGS FOR BOYS AND GIRLS

Looking for adapted solutions



Urinocol® Premature



For baby from
1200 g to 2500 g
Bags for premature
infants

■ 2 sizes: Mini and Standard
depending on baby weight.

« Parachute » shape
to maintain the bag
more effectively
in place.

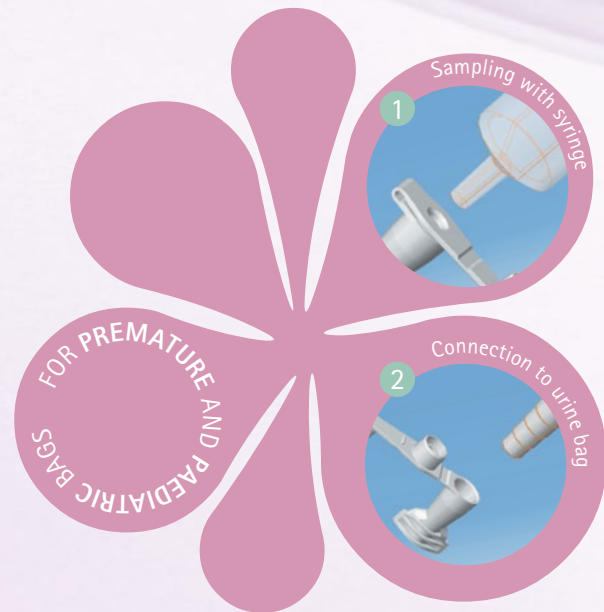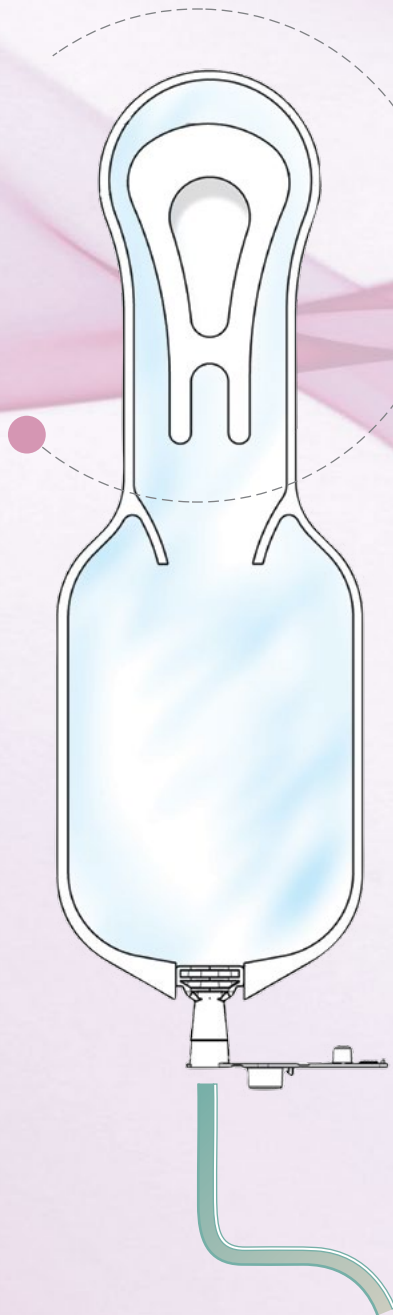


With extension line:
connection to urine bag
adapted for incubator.

Urinocol® Paediatric



For baby > 2 500 g

- High skin tolerance adhesive: gentle on application and removal.
- Adhesive shape adapted to boy and girl anatomy.
- Transparent: permanent visual control of urine.
- Material: PVC free pouch / Latex free adhesive.

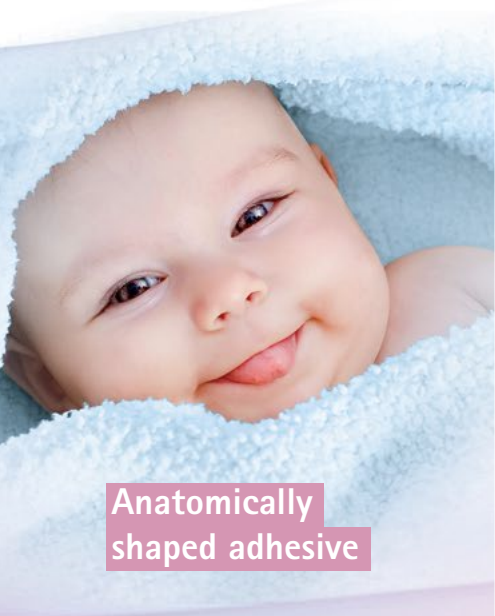

Anatomically shaped adhesive



Adaptation for girl and boy anatomy.



NO DRAINAGE

Closed bag

- Urine sampling for biochemical, cytological or bacteriological examination.



With DRAINAGE

Opened bag

- Collecting bag for diuresis management.


Urinocol® Paediatric

Application step by step for girls:

- Choose the right size! Make sure that the size of the pouch is adapted to the weight of the child.
- Skin preparation: dry, free of oily components.
- Follow the application procedure.



Remove and discard central panel of the adhesive.
Remove the inferior part of the release paper.

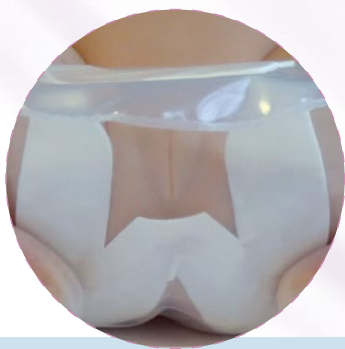

Apply with gentle pressure beneath the baby's penis or vulva.







Remove the superior part of the release paper.




Apply with gentle pressure around the baby's penis or vulva.

Urinocol[®] Paediatric Premature

Urinocol Paediatric ordering information

		Reference	Units/ box
 	Sterile open paediatric bags	Boy 227546 Girl 227566	50 50
 	Sterile closed paediatric bags	Boy 227550 Girl 227560	100 100

Urinocol Premature ordering information

		Weight	Reference	Units/ box
	Sterile premature bags	Standard 1200– 2500 g Mini 1200 g	227571 227570	50 50
	Sterile premature bags (with extension line*)	Standard 1200– 2500 g Mini 1200 g	227573 227572	50 50

*made of PVC: DEHP free



 **Order here!**

 **PRAXISDIENST**
Medical Supplies since 1953

This document, its contents, including institutional data, information, trademarks and logos mentioned herein are the exclusive property of B. Braun. Any representation and / or reproduction, total or partial, of this document and its contents without the express prior consent of B. Braun is strictly prohibited and constitutes an infringement of the intellectual property rights of B. Braun.

Non-binding documents and photographs.

B. Braun Medical declines all liability for the use of this support and particularly for damage suffered by the user resulting from the use of this support or the impossibility of using it

Commercialized and manufactured by:

B. Braun Medical SAS | 204 avenue du Maréchal Juin | 92100 Boulogne Billancourt - France

Tel. 01 41 10 53 00 | Fax 01 41 10 53 99 | www.bbraun.com/continence-care-and-urology

B. Braun Medical SAS with a share capital of 31 000 000 € | RCS Nanterre 562050856