Instruction for use


Ambu[®] AuraGain[™]

Single Use Laryngeal Mask - Sterile


For use by medical professionals trained in airway management only


1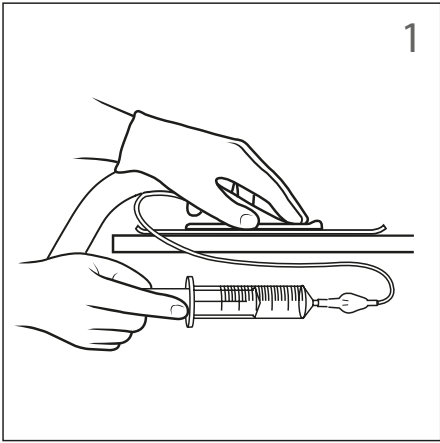


2

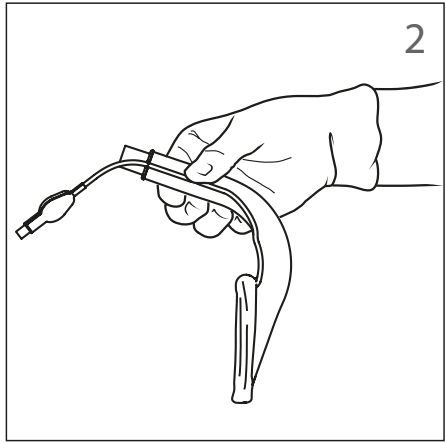
3

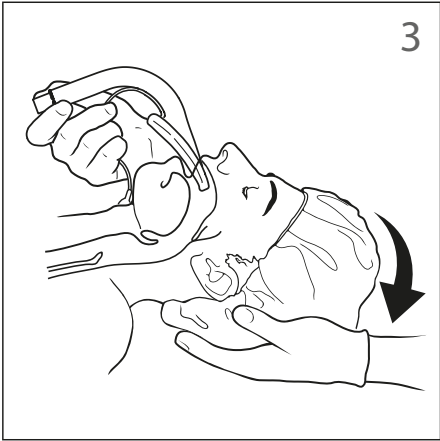
4

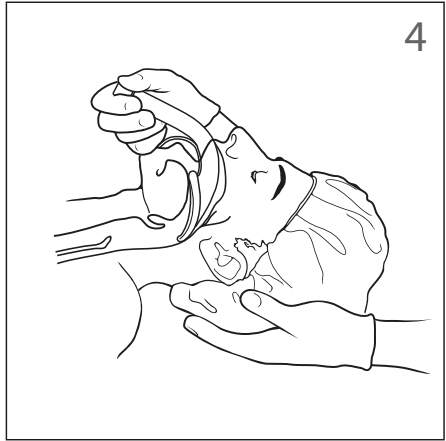5

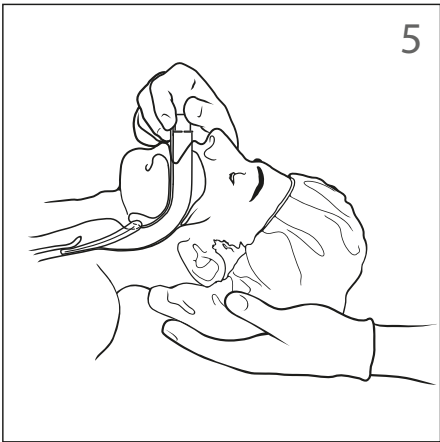6

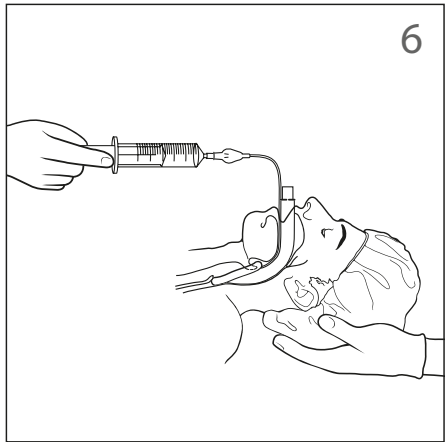Content

Page

English.....	4
България	5
Česky	6
Dansk.....	7
Deutsch	8
Ελληνικά	9
Español.....	10
Eesti.....	11
Suomi.....	12
Français	13
Hrvatski.....	14
Magyar	15
Italiano.....	16
日本語.....	17
Lietuviškai.....	18
Latviski.....	19
Nederlands	20
Norsk.....	21
Polski.....	22
Português.....	23
Romania	24
Русский	25
Slovenčina.....	26
Slovenščina.....	27
Svenska	28
Türkçe	29
中文.....	30

1. Intended use

The Ambu AuraGain is intended for use as an alternative to a face mask for achieving and maintaining control of the airway during routine and emergency anaesthetic procedures.

Intended users: "Professionals trained in airway management".

Have a spare AuraGain ready and prepared for immediate use.

Contents sterile unless packaging is damaged or opened.

WARNINGS

The user should be familiar with the following warnings prior to use of the Ambu AuraGain.

- Do not use the AuraGain before Pre check: Visual inspection, air leak test of the device and compatibility with ET tubes and gastric tubes. Failure of any test indicates that the device should not be used.
- All Ambu AuraGain masks must, prior to use, be investigated and inspected for any potential foreign body. The cuff-protector only serves as protecting the AuraGain during storage and transportation and must be removed prior to use.
- Do not use the Ambu AuraGain if there are any bulges on the cuff or if there are any signs of leakage.
- To avoid trauma, do not use excessive force.
- The Ambu AuraGain is flammable in the presence of lasers and electrocautery equipment.
- For single use only. Use on other patients can cause cross infection. Do not soak, rinse, or sterilize this device as these procedures may leave harmful residues or cause malfunction of the device. The design and material used are not compatible with conventional cleaning and sterilization procedures.
- The presence of a gastric channel does not rule out the possibility of aspiration even if the device is correctly located and fixed in place.
- Do not attempt to pass a gastric tube into the stomach via the gastric channel in the presence of known suspected oesophageal pathology.

CAUTION

- US federal law restricts this device to be sold to or on the order of a physician.
- Ensure that the device is not in any way damaged before use.

For further product information see www.ambu.com

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Ambu[®] is registered trademark of Ambu A/S, Denmark.
Ambu A/S is certified according to ISO 13485.
Ambu A/S