



Disinfection wipes

DESCOCEPT SPEZIAL WIPES

Alcohol-free wipes for rapid disinfection

- Universal, comfortable and timesaving application in a usual dispenser bucket
- Basic, rapid efficacy
- Especially suited for disinfection of large surfaces in routine areas



Product description

DESCOCEPT SPEZIAL WIPES are alcohol-free, ready-to-use Wipes for rapid disinfection and cleaning of medical devices and medical inventory as well as all kind of surfaces in all areas of the routine.

DESCOCEPT SPEZIAL WIPES offer a universal and time saving application for the disinfection and cleaning of large surfaces. The wipes can be extracted individually from the wipes dispenser system with the re-sealable lid. DESOTEX(R) preparation wipes are provided with every refill package to enable a manual and hygienic preparation before refilling the wipes dispenser.

Range of application

According to Biocidal Products Regulation (BPR): For rapid disinfection and cleaning of alcohol sensitive medical equipment and all kind of surfaces.

Acc. to EU Medical Devices Regulation: For rapid disinfection and cleaning of alcohol sensitive, non invasive medical devices.

Application

Wipe surfaces with DESCOCEPT SPEZIAL WIPES until completely wet and allow to dry for the entire contact time.

Spectrum of efficacy and contact times			30 s	1 min	2 min
Bacteria and yeasts					
application recommendation for surface disinfection: bactericidal, yeasticidal*	VAH** / EN 16615	with mechanical action, dirty conditions		•	
Enveloped viruses					
active against enveloped viruses	RKI/DVV	incl HBV/HIV/HCV	•		
	EN 14476	clean conditions	•		
Non-enveloped viruses					
active against papova/SV40 viruses	EN 14476	dirty conditions			•
active against rotaviruses	RKI/DVV		•		

* including phase 2 step 1 and phase 2 step 2 tests (quantitative suspension tests and quantitative carrier tests)
 ** listed in the 5 min column of the VAH list

EN standards			1 min
Antimicrobial properties			
bactericidal	EN 1040	basis test	•
yeasticidal (C. albicans)	EN 1275	basis test	•
bactericidal (medical area)	EN 13727	dirty conditions	•
yeasticidal (C. albicans, medical area)	EN 13624	dirty conditions	•
bactericidal (food and industrial area)	EN 1276 / EN 13697	dirty conditions	•
yeasticidal (C. albicans, food and industrial area)	EN 1650 / EN 13697	dirty conditions	•



Disinfection wipes

DESCOCEPT SPEZIAL WIPES

Alcohol-free wipes for rapid disinfection

Application notes

Shelf life after opening: 28 days.

Composition

100 g solution contain:

0.5 g Alkyl(C12-16)dimethylbenzyl ammonium chloride,

0.25 g didecyldimethylammoniumchloride.

Expert opinions

We have extensive expert reports for all our products. Upon request we are glad to supply you with an expert report portfolio.

Precautionary and hazard statements

Harmful to aquatic life with long lasting effects. Avoid release to the environment. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Dispose of contents/container to approved disposal company or local collection.

For professional use only.

Product	Single unit	Content	Delivery unit	Size	REF
DESCOCEPT SPEZIAL WIPES	Dispenser	100 Wipes	1	32 x 30 cm	00-208-ERS100EXP
	Refill	100 Wipes	2	32 x 30 cm	00-208-RS100EXP

CE 0482



The availability of the products and container sizes depend on a successful national registration.

Use disinfectants safely. Always read label and product information before use.

Dr. Schumacher is certified according to DIN EN 13485, DIN EN ISO9001, DIN EN ISO14001, BS OHSAS 18011 and has a validated environment management system according to EMAS and is a member of IHO, VCI, BAH, DGSV and of the DGKH. www.schumacher-online.com

