

# **Digital Blood Pressure Monitor**

# Model UA-767 Plus BT-Ci

Instruction Manual

Original

Manuel d'instructions

Traduction

Manual de Instrucciones

Traducción

Manuale di Istruzioni

Traduzione

Bedienungsanleitung

Übersetzung

ITALIANO

**ESPAÑOL** 

**ENGLISH** 

DEUTSCH



1WMPD4003087

# Contents

Dear Customers	2
Preliminary Remarks	2
Precautions	2
Parts Identification	4
Symbols	5
Using the Monitor	7
Installing / Changing the Batteries	7
Connecting the Air Hose	7
Connecting the AC Adapter	7
Wireless Function	8
<i>Bluetooth</i> ® Transmission	
Pairing / Unpairing	
Selecting the Correct Cuff Size	
Applying the Arm Cuff	
How to Take Accurate Measurements	
Measurement	
After Measurement	
Measurements	
Normal Measurement	
Measurement with the Desired Systolic Pressure	
Notes for Accurate Measurement	
What is an Irregular Heartbeat	15
Pressure Bar Indicator	15
About Blood Pressure	16
What is Blood Pressure?	16
What is Hypertension and How is it Controlled?	16
Why Measure Blood Pressure at Home?	
WHO Blood Pressure Classification	
Blood Pressure Variations	16
Troubleshooting	17
Maintenance	18
Technical Data	18

### **Dear Customers**

Congratulations on purchasing a state-of-the-art A&D blood pressure monitor, one of the most advanced monitors available today. Designed for ease of use and accuracy, this monitor will facilitate your daily blood pressure regimen.

We recommend that you read through this manual carefully before using the device for the first time.

The *Bluetooth*® word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. and any use of such marks by A&D is under license. Other trademarks and trade names are those of their respective owners.


### **Preliminary Remarks**

This device conforms to the European Directive 93/42 EEC for Medical

Products. This is made evident by the Ceo123 mark of conformity.
(0123: The reference number to the involved notified body)
The WML-40AH is approved in accordance to R&TTE directive transmitter module marked by <b>C</b> €0078 <b>①</b> , manufactured by MITSUMI ELECTRIC CO.,
LTD. to OEM product.
The device complies with part 15 of the FCC rules and contains the FCC ID POOWML-C40.
Compliance with Industry Canada. IC: 4250A-WMLC40.
The device is a Continua certified, <i>Bluetooth®</i> wireless technology enabled medical device.
The device is designed for use on adults only, not newborns or infants.
Environment for use
The device is for use in the home healthcare environment.
This device is designed to measure blood pressure and pulse rate of people
for diagnosis.
Precautions
Precautions  Precision components are used in the construction of this device. Extremes
Precautions  Precision components are used in the construction of this device. Extremes in temperature, humidity, direct sunlight, shock or dust should be avoided. Clean the device and cuff with a dry, soft cloth or a cloth dampened with water and a neutral detergent. Never use alcohol, benzene, thinner or other
Precautions  Precision components are used in the construction of this device. Extremes in temperature, humidity, direct sunlight, shock or dust should be avoided. Clean the device and cuff with a dry, soft cloth or a cloth dampened with water and a neutral detergent. Never use alcohol, benzene, thinner or other harsh chemicals to clean the device or cuff.
Precision components are used in the construction of this device. Extremes in temperature, humidity, direct sunlight, shock or dust should be avoided. Clean the device and cuff with a dry, soft cloth or a cloth dampened with water and a neutral detergent. Never use alcohol, benzene, thinner or other harsh chemicals to clean the device or cuff.  Avoid tightly folding the cuff or storing the hose tightly twisted for long
Precision components are used in the construction of this device. Extremes in temperature, humidity, direct sunlight, shock or dust should be avoided. Clean the device and cuff with a dry, soft cloth or a cloth dampened with water and a neutral detergent. Never use alcohol, benzene, thinner or other harsh chemicals to clean the device or cuff.  Avoid tightly folding the cuff or storing the hose tightly twisted for long periods, as such treatment may shorten the life of the components. The device and cuff are not water resistant. Prevent rain, sweat and water
Precision components are used in the construction of this device. Extremes in temperature, humidity, direct sunlight, shock or dust should be avoided. Clean the device and cuff with a dry, soft cloth or a cloth dampened with water and a neutral detergent. Never use alcohol, benzene, thinner or other harsh chemicals to clean the device or cuff. Avoid tightly folding the cuff or storing the hose tightly twisted for long periods, as such treatment may shorten the life of the components.

	Measurements may be distorted if the device is used close to televisions, microwave ovens, cellular telephones, X-ray or other devices with strong
	electrical fields. Used equipment, parts and batteries are not treated as ordinary household
	waste, and must be disposed of according to the applicable local regulations.
	When the AC adapter is used, make sure that the AC adapter can be readily
_	removed from the electrical outlet when necessary.
	When reusing the device, confirm that the device is clean.
	Do not modify the device. It may cause accidents or damage to the device.
	To measure blood pressure, the arm must be squeezed by the cuff hard
	enough to temporarily stop blood flow through the artery. This may cause
	pain, numbness or a temporary red mark to the arm. This condition will
	appear especially when measurement is repeated successively.
	Any pain, numbness, or red marks will disappear with time.
	Take care to avoid accidental strangulation of babies or infants with the hose.
_	Do not twist the air hose during measurement. This may cause injury due to continuous cuff pressure.
	Measuring blood pressure too frequently may cause harm due to blood flow
_	interference. Check that the operation of the device does not result in
	prolonged impairment of blood circulation, when using the device repeatedly.
	If you have had a mastectomy, please consult a doctor before using the device.
	Do not let children use the device by themselves and do not use the device
_	in a place within the reach of infants.
	There are small parts that may cause a choking hazard if swallowed by
	mistake by infants.
	Wireless communication devices, such as home networking devices, mobile
	phones, cordless phones and their base stations, walkie-talkies can affect
	this blood pressure monitor.
	Therefore, a minimum distance of 11 feet (3.3 meters) should be kept from
	such devices.
	ntraindications
	following are precautions for proper use of the device.
	Do not apply the cuff to an arm with another medical electrical equipment
_	attached. The equipment may not function properly.
	People who have a severe circulatory deficit in the arm must consult a doctor
	before using the device, to avoid medical problems.
	Do not self-diagnose the measurement results and start treatment by
	yourself. Always consult your doctor for evaluation of the results and treatment.
	Do not apply the cuff on an arm with an unhealed wound.
	Do not apply the cuff on an arm receiving an intravenous drip or blood
_	transfusion. It may cause injury or accidents.
	Do not use the device where flammable gases such as anesthetic gases
_	are present. It may cause an explosion.
	Do not use the device in highly concentrated oxygen environments, such
_	as a high-pressure oxygen chamber or an oxygen tent. It may cause a fire
	or explosion.

### **Parts Identification**

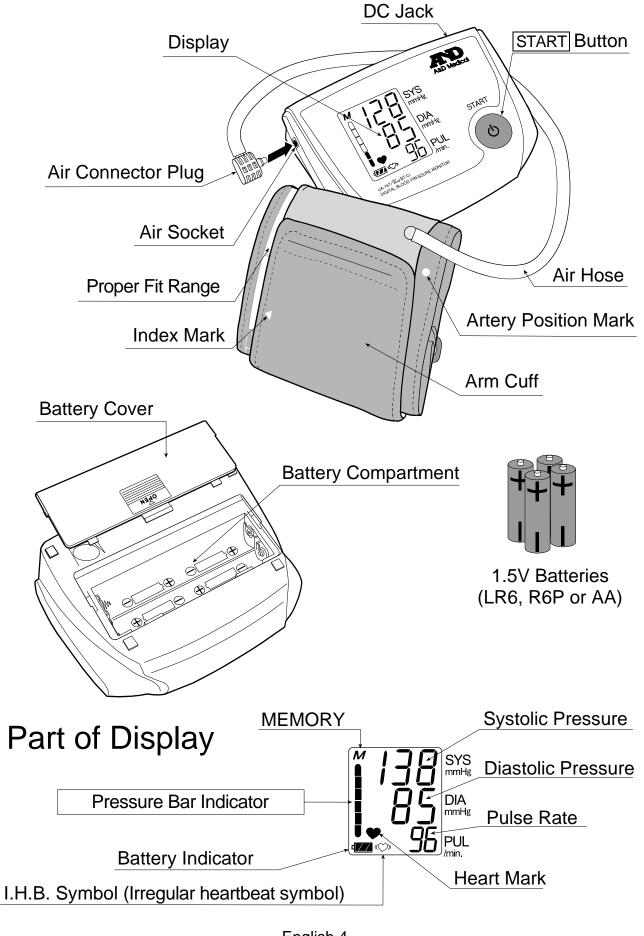

English 4


# **Symbols**

Symbols	Function / Meaning	Recommended Action
Ů	Standby and power on.	
$\oplus$	Battery installation guide	
	Direct current	
	Type BF: Device, cuff and tubing are designed to provide special protection against electrical shocks.	
•	Appears while measurement is in progress. It blinks when the pulse is detected.	Measurement is in progress. Remain as still as possible.
((\(\sigma\))	Irregular <b>H</b> eart <b>b</b> eat symbol (I.H.B.) Appears when an irregular heartbeat is detected. It may light when a very slight vibration like shivering or shaking is detected.	·
M	Previous measurements stored in MEMORY.	
	FULL BATTERY The battery power indicator during the measurement.	
	LOW BATTERY The indicator blinks when battery power is low.	Replace all batteries with new ones, when the indicator blinks.
	Unstable blood pressure due to movement during the measurement.	Try the measurement again. Remain very still during the measurement.
	The systolic and diastolic values are within 10 mmHg of each other.	
	The pressure value did not increase during inflation.	Fasten the cuff correctly, and try the measurement
LUF	The cuff is not fastened correctly.	again.
Err	PUL DISPLAY ERROR The pulse is not detected correctly.	
Err 10	Pairing or unpairing has not been	Remove and re-insert the batteries. Try pairing or
Err	performed correctly.	unpairing again.
Pr	Pairing in progress	
uPr	Unpairing in progress	
End	Pairing or unpairing complete	
SYS	Systolic blood pressure in mmHg	
DIA	Diastolic blood pressure in mmHg	
PUL	Pulse per minute	

Symbols	Function / Meaning	Recommended Action
<b>C</b> € <sub>0123</sub>	EC directive medical device label ————	
<b>C €</b> 0678 <b>①</b>	R&TTE directive transmitter module label ————	
<b></b>	Manufacturer	
EC REP	EU-representative	
2013سا	Date of manufacture	
	WEEE label	
<b>③</b>	Refer to instruction manual/booklet	
SN	Serial number	
⊖ <del>.</del> €⊕	Polarity of DC jack	
IP	International protection symbol	

#### **Installing / Changing the Batteries**

- 1. Slide the battery cover up to open it.
- 2. Remove the used batteries and insert new batteries into the battery compartment as shown, taking care that the polarities (+) and (-) are correct.
- 3. Slide the battery cover down to close. Use only LR6, R6P or AA batteries.

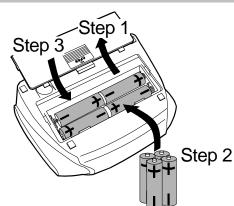


#### **CAUTION**

- Insert the batteries as shown in the battery compartment. If installed incorrectly, the device will not work.
- □ When □ (LOW BATTERY mark) blinks in the display, replace all batteries with new ones. Do not mix old and new batteries. It may shorten the battery life, or cause the device to malfunction.
- □ □ (LOW BATTERY mark) does not appear when the batteries are drained.
- The battery life varies with the ambient temperature and may be shorter at low temperatures. Generally, four new LR6 batteries will last approximately for six months when used twice for measurement each day.
- Remove the batteries if the device is not to be used for a long time. The batteries may leak and cause a malfunction.
- ☐ Use the specified batteries only. The batteries provided with the device are for testing monitor performance and may have a limited life.

#### **Connecting the Air Hose**


Insert the air connector plug into the air socket firmly.

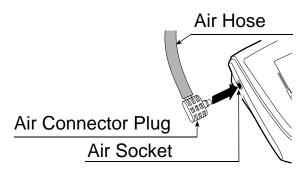

#### Connecting the AC Adapter

Insert the AC adapter plug into the DC jack. Next, connect the AC adapter to an electrical outlet.

- ☐ Use the specified AC adapter. (Refer to page 19.)
- □ When disconnecting the AC adapter from the electrical outlet, grasp and pull the AC adapter body out of the outlet.
- ☐ When disconnecting the AC adapter plug from the blood pressure monitor, grasp and pull the AC adapter plug out of the monitor.

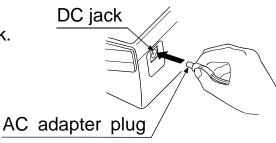

# Wireless Function CAUTION

- ☐ In the unlikely event that this monitor causes radio wave interference in a different wireless station, change the location where this monitor is used or stop use immediately.
- □ Be sure to use in a location where visibility between the two devices that you want to connect is good. The connection distance is reduced by the structure of buildings or other obstructions. In particular, connection may be impossible when devices are used on either side of reinforced concrete.
- Do not use Bluetooth® connection in the range of a wireless LAN or other wireless devices, near devices that emit radio waves such as microwaves, in locations where there are many obstructions, or in other locations where signal strength is weak. Doing so may result in frequent loss of connection, very slow communication speeds and errors.
- □ Using close to an IEEE802.11g/b/n wireless LAN device may cause mutual interference to occur, which may result in reduced communication speeds or which may prevent connection.
  - In this case, switch off the power supply to the device that is not being used, or use the monitor in a different location.
- ☐ If the monitor does not connect normally when used near a wireless station or broadcast station, use the monitor in a different location.
- □ A&D Company, Limited cannot accept liability for any damages incurred due to impaired operation or data loss, etc. that occur through the use of this product.
- ☐ This product is not guaranteed to connect to all *Bluetooth*® compatible devices.

#### **Bluetooth®** Transmission

This product is equipped with a *Bluetooth®* wireless function and can connect to the following *Bluetooth®* devices.

Bluetooth® devices that can be connected:

- Continua certified devices
- iPhone, iPad, iPod
- Applications and devices that are compatible with SSP and A&D specifications

Each device needs an application to receive data.

For connection methods, refer to the manual for each device.



Bluetooth® devices carry the Bluetooth® logo mark.



Continua certified devices carry the Continua logo mark.



iPhone, iPad and iPod are trademarks of Apple Inc., registered in the U.S. and other countries.

#### Pairing / Unpairing

A *Bluetooth®* device needs to be paired with a different specific device in order to communicate with that device. If this monitor is paired with a receiver device from the start, measurement data is transmitted automatically to the receiver device each time a measurement is made.

Unpairing needs to be performed to cancel pairing.

#### Cautions for pairing

Be sure to turn the power of all other <i>Bluetooth®</i> devices off when pairing.
Multiple devices cannot be paired at the same time.
If the receiver device cannot receive measurement data, try pairing once
again.
Two receiver devices can be paired to one monitor. Communication is

possible with one of the receiver devices.

If a third receiver device is paired, the device other than the most recently connected device will be unpaired to enable the new device to be paired.

□ Some applications may allow pairing with only one device.

Follow the steps below to pair or unpair the monitor with a *Bluetooth*® compatible receiver device. Also refer to the manual of the receiver device. Please use a pairing wizard if one is provided.

#### Pairing procedure

1. Follow the instructions in the manual of the receiver device to switch it to the pairable status. When pairing this monitor, place it as close as possible to the receiver device to be paired with.

Pr

- Install the batteries or connect the AC adapter as described on page 7.
   The monitor displays "Pr".
   The monitor will be in a state that can be found by the receiver device for about two minutes.
- receiver device for about two minutes.

  3. The monitor displays "End" to indicate that pairing is complete.
- 4. If "Err II" is displayed or pairing is failed, remove the batteries or disconnect the AC adapter and try steps 1-3 again.
- 5. Follow the manual of the pairing receiver device to search for, select and pair with this monitor.

  If a PIN code is requested by the receiver device, enter "39 12 1440".

#### **Unpairing procedure**

1. Press and hold the START button until "uPr" is displayed.

- ٦٩٢
- 2. Press the START button again. Unpairing will be performed.
- 3. The monitor displays "End" to indicate that unpairing is complete.

  When two receiver devices are paired, both devices



4. If "Err ID" or "Err I I" is displayed or unpairing is failed, try steps 1-3 again.

#### **Communication distance**

are unpaired.

The communication distance between this monitor and the receiver device is dependent on the *Bluetooth®* output class of the receiver device.

When the receiver device is a Class 1 *Bluetooth®* device: Less than 100 m When the receiver device is a Class 2 *Bluetooth®* device: Less than 10 m

This distance is reduced by the conditions in the surrounding environment, so be sure to check that the distance is short enough for a connection to be made after measurement is complete.

#### Memory

In cases when the receiver device cannot receive measurement data, the measurement data is temporarily stored in the monitor memory. The data stored in the memory is transmitted the next time a connection is successfully made to the receiver device.

A total of 200 sets of measurement data can be stored. When the amount of data exceeds 200 sets, the oldest data is deleted and the new data is stored. The amount of data that can be stored temporarily may vary with the application.

#### **Time**

This monitor has a built-in clock. The date and time that a measurement was taken is included in the measurement data.

The built-in clock is designed to be automatically adjusted by syncing with the clock of a receiver device. Refer to the specifications of the receiver device.

This monitor has no clock adjustment function.

#### **Selecting the Correct Cuff Size**

Using the correct cuff size is important for an accurate reading. If the cuff is not the proper size, the reading may yield an incorrect blood pressure value.


- ☐ The arm size is printed on each cuff.
- ☐ The index ▲ and proper fit range, on the cuff, tell you if you are applying the correct cuff. (Refer to "Symbols that are printed on the cuff".)
- ☐ If the index ▲ points outside of the range, contact your local dealer to purchase a replacement cuff.
- ☐ The arm cuff is a consumable. If it becomes worn, purchase a new one.

Arm Size	Recommended Cuff Size	Catalog Number
14.2" to 17.7" / 36 cm to 45 cm	Large cuff	UA-281
9.4" to 14.2" / 24 cm to 36 cm	Medium cuff	UA-280
6.3" to 9.4" / 16 cm to 24 cm	Small cuff	UA-279

Arm size: The circumference of the biceps.


#### Symbols that are printed on the cuff

Symbols	Function/Meaning	Recommended Action
•	Artery position mark	Set the mark on the artery of the upper arm or in line with the ring finger on the inside of the arm.
	Index	
REF	Catalog number	
M	Proper fit range for the medium cuff. It's printed on the medium cuff.	
	Over range printed on the medium cuff.	Use the large cuff instead of the medium cuff.
	Proper fit range for the large cuff. It's printed on the large cuff.	
S	Under range printed on the medium cuff.	Use the small cuff instead of the medium cuff.
S	Proper fit range for the small cuff. It's printed on the small cuff.	
$\overline{M}$	Under range printed on the large cuff.  Over range printed on the small cuff.	Use the medium cuff instead of the large cuff or small cuff.
LOT	Lot number	

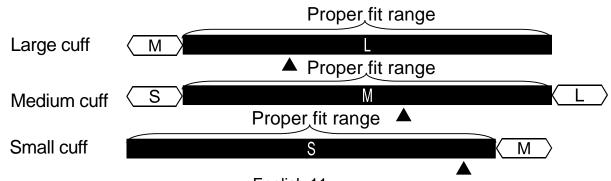

#### **Applying the Arm Cuff**

- Wrap the cuff around the upper arm, about 1-2 cm above the inside of the elbow, as shown.
  - Place the cuff directly against the skin, as clothing may cause a faint pulse, and result in a measurement error.
- Constriction of the upper arm, caused by tightly rolling up a shirtsleeve, may prevent accurate readings.
- 3. Confirm that the index ▲ points within the proper fit range.

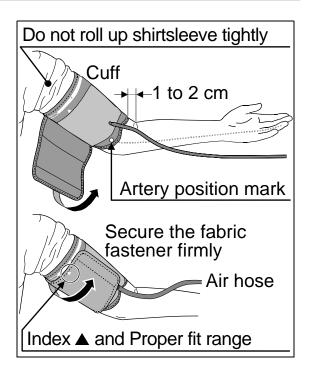

#### **How to Take Accurate Measurements**

For the most accurate blood pressure measurement:

- □ Sit comfortably on a chair. Rest your arm on the table. Do not cross your legs. Keep your feet on the floor and straighten your back.
- □ Relax for about five to ten minutes before measurement.
- □ Place the center of the cuff at the same height as your heart.
- □ Remain still and keep quiet during measurement.
- Do not measure immediately after physical exercise or a bath. Rest for twenty or thirty minutes before taking the measurement.
- ☐ Try to measure your blood pressure at the same time every day.

#### Measurement

During measurement, it is normal for the cuff to feel very tight. (Do not be alarmed)

#### **After Measurement**

After measurement, press the START button to turn off the power. Remove the cuff and record your data.

Note: The device has an automatic power shut-off function, which turns the power off approximately one minute after measurement.

Allow at least three minutes between measurements on the same person.

### **Measurements**

Model UA-767 Plus BT-Ci is designed to detect the pulse and to inflate the cuff to a systolic pressure level automatically.

If your systolic pressure is expected to exceed 230 mmHg, read "Measurement with the Desired Systolic Pressure" on the next page.

#### **Normal Measurement**

- Place the cuff on the arm (preferably the left arm).
   Sit quietly during measurement.
- 2. Press the START button.


  The last data of systolic and diastolic pressure and pulse rate are displayed briefly. Then the display changes, as indicated in the figure at the right, as the measurement begins. The cuff starts to inflate. It is normal for the cuff to feel very tight. A pressure bar indicator is displayed, as in the figure at the right, during inflation.

Note: If you wish to stop inflation at any time, press the START button again.

3. When inflation is complete, deflation starts automatically and the ♥ (heart mark) blinks, indicating that the measurement is in progress. Once the pulse is detected, the mark blinks with each pulse beat.

Note: If an appropriate pressure is not obtained, the device starts to inflate again automatically.

- 4. When the measurement is complete, the systolic and diastolic pressure readings and pulse rate are displayed. The cuff exhausts the remaining air and deflates completely.
- 5. Press the START button again to turn off the power.

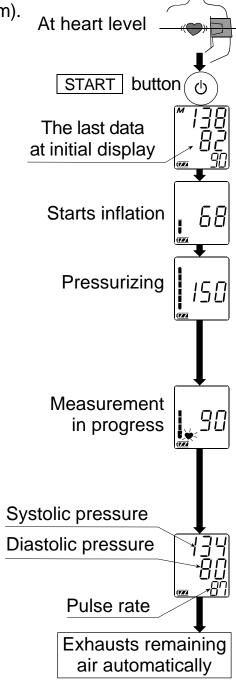


Note: Model UA-767 Plus BT-Ci is provided with an automatic power shut-off function. Allow at least three minutes between measurements on the same person.

### **Measurements**

#### **Measurement with the Desired Systolic Pressure**

If your systolic pressure is expected to exceed 230 mmHg, use this procedure.

- 1. Place the cuff on the arm (preferably the left arm).
- 2. Press the START button.
- 3. While the zero blinks, press and hold the START button until a number about 30 to 40 mmHg higher than your expected systolic pressure appears.
- 4. When the desired number is reached, release the START button to start measurement. Then continue to measure your blood pressure as described on the previous page.

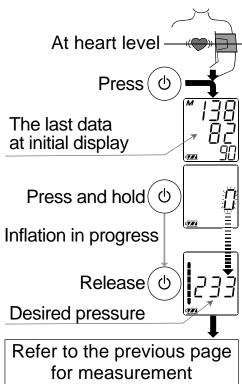

#### **Notes for Accurate Measurement**

- Sit down in a comfortable position. Place your arm on a table with your palm facing upward and the cuff at the same level as your heart.
- Relax for about five to ten minutes before taking a measurement. If you are excited or depressed by emotional stress, the measurement will reflect this stress as a higher (or lower) than normal blood pressure reading and the pulse reading will usually be faster than normal.
- An individual's blood pressure varies constantly, depending on what you are doing and what you have eaten. What you drink can have a very strong and rapid effect on your blood pressure.
- ☐ This device bases its measurements on the heartbeat. If you have a very weak or irregular heartbeat, the device may have difficulty determining your blood pressure.
- □ Should the device detect a condition that is abnormal, it will stop the measurement and display an error symbol. See page 5 for the description of symbols.
- ☐ This blood pressure monitor is intended for use by adults only. Consult with your physician before using this device on a child. A child should not use this device unattended.


## What is an Irregular Heartbeat

Model UA-767 Plus BT-Ci blood pressure monitor provides a blood pressure and pulse rate measurement even when an irregular heartbeat occurs. An irregular heartbeat is defined as a heartbeat that varies by 25% from the average of all heartbeats during the blood pressure measurement. It is important that you be relaxed, remain still and do not talk during measurements.

Note: We recommend contacting your physician if you see this (S) symbol frequently.

### **Pressure Bar Indicator**

The indicator monitors the progress of pressure during measurement.

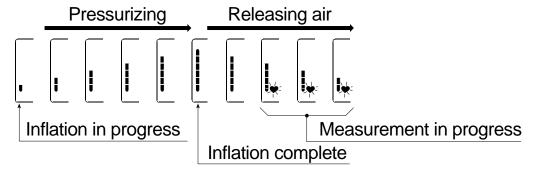

### **About Blood Pressure**

#### What is Blood Pressure?

Blood pressure is the force exerted by blood against the walls of the arteries. Systolic pressure occurs when the heart contracts. Diastolic pressure occurs when the heart expands. Blood pressure is measured in millimeters of mercury (mmHg). One's natural blood pressure is represented by the fundamental pressure, which is measured first thing in the morning while one is still at rest and before eating.

#### What is Hypertension and How is it Controlled?

Hypertension, an abnormally high arterial blood pressure, if left unattended, can cause many health problems including stroke and heart attack. Hypertension can be controlled by altering lifestyle, avoiding stress, and with medication under a doctor's supervision.

To prevent hypertension or keep it under control:

□ Do not smoke		Exercise	regularly
----------------	--	----------	-----------

□ Reduce salt and fat intake □ Have regular physical check
--


□ Maintain proper weight

#### Why Measure Blood Pressure at Home?

Blood pressure measured at a clinic or doctor's office may cause apprehension and can produce an elevated reading, 25 to 30 mmHg higher than that measured at home. Home measurement reduces the effects of outside influences on blood pressure readings, supplements the doctor's readings and provides a more accurate, complete blood pressure history.

#### **WHO Blood Pressure Classification**

Standards to assess high blood pressure, without regard to age, have been established by the World Health Organization (WHO), as shown in the chart.

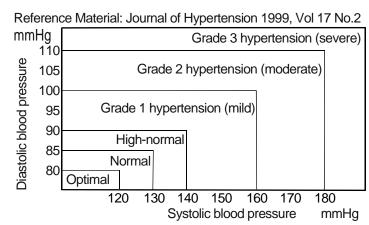


#### **Blood Pressure Variations**

An individual's blood pressure varies greatly on a daily and seasonal basis. It

may vary by 30 to 50 mmHg due to various conditions during the day. In hypertensive individuals, variations are even more pronounced. Normally, the blood pressure rises while at work or play and falls to its lowest levels during sleep. So, do not be overly concerned by the results of one measurement.

Take measurements at the same time every day using the procedure described in this manual to get to know your normal blood pressure. Regular readings give a more comprehensive blood pressure history. Be sure to note date and time when recording your blood pressure. Consult your doctor to interpret your blood pressure data.

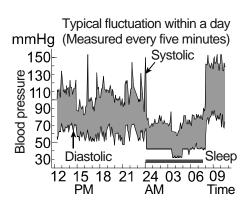

### **Troubleshooting**

Problem	Possible Reason	Recommended Action
Nothing appears	Batteries are drained.	Replace all batteries with new ones.
in the display, even when the power is turned on.	Battery terminals are not in the correct position.	Reinstall the batteries with negative and positive terminals matching those indicated on the battery compartment.
The cuff does not inflate.	Battery power is low.  (LOW BATTERY mark) blinks.  If the batteries are drained completely, the mark does not appear.	Replace all batteries with new ones.
	The cuff is not applied properly.	Apply the cuff correctly.
The unit does	You moved your arm or body during the measurement.	Make sure you remain very still and quiet during the measurement.
not measure. Readings are too high or too low.	The cuff position is not correct.	Sit comfortably and still. Place your arm on a table with your palm facing upward and the cuff at the same level as your heart.
		If you have a very weak or irregular heartbeat, the device may have difficulty in determining your blood pressure.
Other	The value is different from that measured at a clinic or doctor's office.	See "Why measure blood pressure at home?".
Otilei		Remove the batteries. Place them back properly and try the measurement again.

Note: If the actions described above do not solve the problem, contact the dealer. Do not attempt to open or repair this product, as any attempt to do so will make your warranty invalid.

### **Maintenance**

Do not open the device. It uses delicate electrical components and an intricate air unit that could be damaged. If you cannot fix the problem using the troubleshooting instructions, contact the authorized dealer in your area or our customer service department. The A&D customer service will provide technical information, spare parts and units to authorized dealers.

The device was designed and manufactured for a long service life. However it is generally recommended to have the device inspected every 2 years, to ensure proper functioning and accuracy. Please contact the authorized dealer in your area or A&D for maintenance.

### **Technical Data**

Type UA-767PBT-Ci

Measurement method Oscillometric measurement Measurement range Pressure: 20 - 280 mmHg

Pulse: 40 - 200 beats / minute

Measurement accuracy Pressure: ±3 mmHg

Pulse: ±5%

Power supply 4 x 1.5V batteries (LR6, R6P or AA) or

AC adapter (TB-233) (Not included)

batteries are used, with pressure value of 180

mmHg at room temperature of 23°C

Classification Internally powered ME equipment (Supplied by

batteries) /

Class II (Supplied by adapter)

Clinical test According to ANSI / AAMI SP-10 1987

EMC IEC 60601-1-2: 2007

Wireless communication WML-40AH (MITSUMI ELECTRIC CO., LTD.)

Bluetooth Ver.2.1 Class 1 HDP SSP

Continua certified

Operating conditions +10°C to +40°C / 15%RH to 85 %RH /

800 hPa to 1060 hPa

Transport / Storage conditions -10°C to +60°C / 15%RH to 85 %RH /

700 hPa to 1060 hPa

Dimensions Approx. 147 [W] x 64 [H] x 110 [D] mm

Weight Approx. 300 g, excepting batteries

Applied part Cuff Type BF 🖈

Ingress protection Device: IP20

Useful life Device: 5 years (when used six times a day)

Cuff: 2 years (when used six times a day)

Accessory AC adapter The adapter is to connect the blood pressure

monitor to a power source at home.

TB-233 Please contact your local A&D dealer for

purchasing.

The AC adapter is required to be inspected or

replaced periodically.

TB-233C Input: 100-240 V

Output: 6 V === 500 mA

⊖**€**⊕ □ 🖒 🚍

TB-233BF Input: 100-240 V

Output: 6 V === 500 mA

**○€** □ 🖒 🛱 🚍

TB-233 Input: 120 V

Output: 6 V === 500 mA

**○€** □ 🖒 🛱 ΞΑ

#### Accessories sold separately

#### Cuff

Catalog Number	Cuff Size	Arm Size
UA-281	Large cuff	14.2" to 17.7" / 36 cm to 45 cm
UA-280	Medium cuff	9.4" to 14.2" / 24 cm to 36 cm
UA-279	Small cuff	6.3" to 9.4" / 16 cm to 24 cm

AC adapter

Catalog Number	Plug
TB-233C	Type C
TB-233BF	Type BF
TB-233	Type A

Note: Specifications are subject to change for improvement without prior notice.



#### A&D Company, Limited

1-243 Asahi, Kitamoto-shi, Saitama 364-8585 JAPAN Telephone: [81] (48) 593-1111 Fax: [81] (48) 593-1119

#### EC REP A&D INSTRUMENTS LIMITED

Unit 24/26 Blacklands Way, Abingdon Business Park, Abingdon, Oxfordshire OX14 1DY United Kingdom Telephone: [44] (1235) 550420 Fax: [44] (1235) 550485

#### **A&D ENGINEERING, INC.**

1756 Automation Parkway, San Jose, California 95131 U.S.A. Telephone: [1] (408) 263-5333 Fax: [1] (408)263-0119

#### **A&D AUSTRALASIA PTY LTD**

32 Dew Street, Thebarton, South Australia 5031 AUSTRALIA Telephone: [61] (8) 8301-8100 Fax: [61] (8) 8352-7409